82 to 96 cm	60 cm	43 cm	58 cm	68 cm	18 cm	20 kg	11,2 kg
-------------	-------	-------	-------	-------	-------	-------	---------

FR	EN	DE	ES	IT	NL	SI
1. Signification des symboles	Explanation of symbols	Zichensymbolik	Explicación de los símbolos	Significato dei simboli	Verklaring van de symbolen	Pomen simbolov
2. Lire le mode d'emploi avant d'utiliser le produit.	Read the instruction manual before use	Medienprodukt	Dispositivo médico	Dispositivo medico	Medisch hulpmiddel	Medicinska pripomočka
3. Respecter les instructions de sécurité.	Observe the safety instructions	Vor Gebrauch Gebrauchsanweisung befolgen	Leer el manual de instrucciones antes de utilizar el dispositivo	Prima dell'uso leggere le istruzioni	Voor gebruik de handleiding respecteren	Pred uporabo prebrati navodila za uporabo
4. Conformité CE	CE conformity	CE-Konformität	Conformidad con la normativa CE	CE conformità	CE conformiteit	CE Conformit
1. Significato dei simboli	Ogólna symbolika	Szimbólum jelölés	Símbolos de identificación	Objašnjenje simbola	Erklärungen der Symbole	Pomen simbolov
2. Lire le mode d'emploi avant d'utiliser le produit.	Read the instruction manual before use	Vor Gebrauch Gebrauchsanweisung befolgen	Leer el manual de instrucciones antes de utilizar el dispositivo	Prima dell'uso leggere le istruzioni	Voor gebruik de handleiding respecteren	Pred uporabo prebrati navodila za uporabo
3. Respecter les instructions de sécurité.	Observe the safety instructions	Vor Gebrauch Gebrauchsanweisung befolgen	Leer el manual de instrucciones antes de utilizar el dispositivo	Prima dell'uso leggere le istruzioni	Voor gebruik de handleiding respecteren	Pred uporabo prebrati navodila za uporabo
4. Conformité CE	CE conformity	CE-Konformität	Conformidad con la normativa CE	CE conformità	CE conformiteit	CE Conformit

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- 1. Safety indication**
- ▶ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
 - ▶ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !).
 - ▶ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
 - ▶ Check carefully all parts of this item (screws, tubes, tips, grips...).
 - ▶ Check regularly if all screws are tightened.
 - ▶ Hold firmly and completely the handle when using.
 - ▶ In case of falling : let the product fall down on the side, so that you don't fall on it.
 - ▶ Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
 - ▶ Don't use on wet or icy surfaces (snow, ice, water...)
 - ▶ Don't use in stairs, severe risk of injuries!
 - ▶ Use only as a walking aid and in proper way : don't use as a lever or to move furniture.
 - ▶ ALWAYS observe the weight limit on the labelling of your rollator.
 - ▶ After unfolding or assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before using.
 - ▶ DO NOT hang anything on the Walking aid.
 - ▶ ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
 - ▶ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.