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## **IMPORTANT**

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

## **PRODUCT DESTINATION**

Bermuda toilet armrests are toilet aids for people with limited mobility. These products are intended for indoor use.

## **PRODUCT DESCRIPTION**

## **SAFETY PRECAUTIONS**

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 125 kg.
- Use only the accessories and parts supplied by the manufacturer.
- Visually check the general condition of the frame.
- Make sure that the toilet armrest is correctly unfolded and that all legs are adjusted to the same height.
- Ensure that all parts are secure and in good working order.
- Do not make any adjustments to the toilet armrest while it is in use.
- Do not use the toilet armrest on stairs or escalators.
- Make sure you put your hands on each handle so that the toilet armrest can be in a parallel force position.
- Do not use the toilet armrest as a step support.
- Do not use the toilet armrest to walk backwards.
- Always lean on the toilet armrest.
- Install on a tiled floor for floor suction cups
- Ensure that the clips are fully inserted for height adjustment before each use

- In case of damaged, broken or worn parts, contact the dealer.

## MOUNTING

1. Take all the parts out of the packing box and make sure that all the components are present (see picture below)

2. Insert the legs into the frame by pressing the snap button. Align the pin and the hole until the pin comes out completely. Adjust the height of the legs, making sure that all 4 legs are set to the same height.

3. Assemble the side frames with the two tubes. Attach the tubes to the side frames using the "C" clip. You can adjust the width of the product to your liking.

4. Place the Bermuda Toilet Armrest around your toilet. Insert the toilet support kit on each side. Adjust and secure the kit with the nuts. Make sure the kit is tight around your toilet.

## USE

To use the armrest, place your forearms on the armrests and your hands on the handles. Adjust the height if necessary. Move forward slowly.

## CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Before each use, check that the spring clips are fully extended and the fixing plate tightened at the back of the toilet.
- Check before each use that the fixing plates are firmly attached to the toilet
- Regularly check the condition of the armrests and the tightness of the fixings
- Regularly check the condition of the armrests and the tightness of the fixings.

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place

- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

## FEATURES TECHNICAL

Overall dimensions	W 54.5/59.5 (between armrests 45/50.5) x D 44.5 x H 69/82 cm in 6 positions
Maximum supported weight	125 kg

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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